

# Cleaning instructions

	DAILY	WEEKLY	MONTHLY	QUARTERLY	EVERY HALF YEAR
<b>MASK</b>					
Soft parts of the mask	<ol style="list-style-type: none"> <li>1. Moisten</li> <li>2. Rub with dish soap</li> <li>3. Rinse with water</li> <li>4. Let it dry</li> </ol>				
Hard parts of the mask		<ol style="list-style-type: none"> <li>1. Make a lather of dish soap and water</li> <li>2. Rub clean with a soft brush</li> <li>3. Air-dry</li> </ol>			
Headgear			<ol style="list-style-type: none"> <li>1. Make a lather of mild soap / fine dish soap + water</li> <li>2. Wash the headgear</li> <li>3. Let it dry</li> </ol> <p><u>NO</u> washing machine</p>		
<b>TUBING</b>		<ol style="list-style-type: none"> <li>1. Fill tubing with water + dish soap</li> <li>2. Hold ends closed and shake</li> <li>3. Rinse clean</li> <li>4. Let it dry</li> </ol>			
<b>HUMIDIFIER</b>	<ol style="list-style-type: none"> <li>1. Remove humidifier from device</li> <li>2. Empty and rinse</li> <li>3. Let it dry well</li> </ol>	<ol style="list-style-type: none"> <li>4. Fill humidifier with 1/5 cleaning vinegar and 4/5 water</li> <li>5. Soak for 1 hour</li> <li>6. Empty and brush clean</li> <li>7. Rinse clean</li> </ol>			
<b>FILTERS</b>					
ResMed, white		Whip out		Replace	
Weinmann, white			Replace		
Weinmann, black		Rinse with water			Replace
Fisher & Paykel, white		Whip out		Replace	
Löwenstein, black		Rinse with water			Replace
Philips, black		Rinse with water			Replace
Philips Dreamstation, white		Rinse with water			Replace